DE

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EN I. FOREWORD

These instructions are for use valid for tube holders.

The holder should be used to serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling.

II. INTENDED USE

The tube holder allows the tracheostomy tube to be fixed securely in place and can be adjusted to fit different necks.

III. WARNGUNGEN

NECKFK® are single-patient products and are intended for single use only.

CAUTION!

Cleaning, disinfection, re-sterilisation and re-use can impair the safety and function of the product and are therefore not permissible!

The tubeholder should not be fitted too tightly around the neck. There should ideally be a distance of approx. 1 cm between tube holder and neck.

To prevent a tracheostomy tube from coming off by accident, the tube must be checked regularly to make sure it is still securely fixed in place.

IV. INSTRUCTIONS

NECKFK® DUO

Attach the two parts of the tube holder to each other by means of the adherent fastener, so that the correct length (neck size) is obtained **①**.

NECKFK®/NECKFK® DUO

Please attach the tube holder around your neck with the hook-and-loop fasteners facing outward **②**.

Attach the hook-and-loop fasteners to the lateral eyelets on both sides of the neck flange **③**.

V. SERVICE LIFE

The maximum period of use / wearing period is 24 hours.

VI. LEGAL NOTICES

The manufacturer Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for damages caused by unauthorized product alterations, by repairs not performed by the manufacturer or by incorrect handling, cleaning, (de)sterilization and/or re-use. The manufacturer is not liable for the provision of the product to a third party. Should a serious adverse event occur in connection with this product of Andreas Fahl Medizintechnik-Vertrieb GmbH, this is to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient are domiciled. Product specifications described herein are subject to change without notice, NECKFK® is a trademark and brand of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne, registered in Germany and the EU member states.

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